# lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

shower chair  
with back

silla con respaldo  
para la ducha

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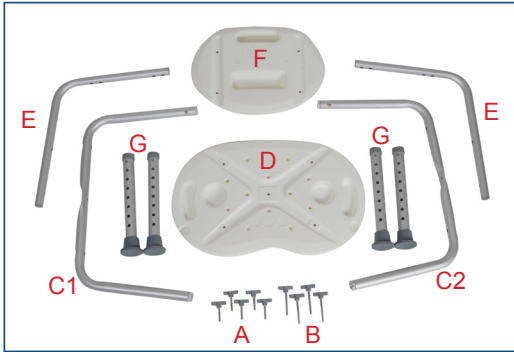
item # RTL12605

# assembly instructions

Weight capacity 300 lbs (136 Kg)

- A- Short Thumb Screw (5)
- B- Large Thumb Screw (4)
- C- Leg Receivers (2)
- D- Seat (1)
- E- Backrest Frame (2)
- F- Backrest (1)
- G- Legs (4)

1. Remove contents from carton and verify that all parts in the parts diagram/list are present.
2. Place seat upside down on a flat surface so holes are showing.
3. Align the leg section that has the concave area on the underside with the holes in seat (Figure 1). Align the leg section that has the concave area on the top side with the leg section below (Figure 2).

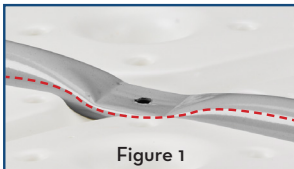
Figure 1

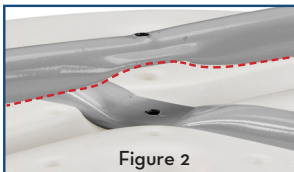
Figure 2

4. Secure in place through the center hole using a SMALL thumb screw (A) (Figure 3).

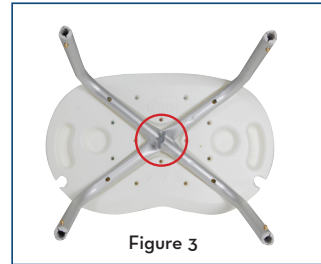
Figure 3

5. Attach backrest frame (E) to the seat by aligning holes on backrest frame supports with the holes on the leg receivers (C). Secure in place using the large thumb screws (B), making sure to pay attention to the “rear” of the seat (Figure 4).

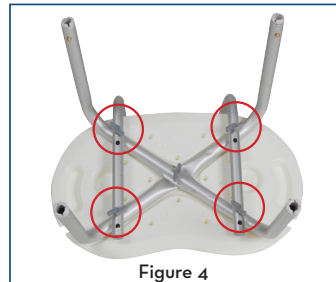
Figure 4

6. Insert legs (G) into leg frames by depressing push pins, sliding legs over the receptacle and setting to desired height. Make sure all the legs are set to the same height (Figure 5).

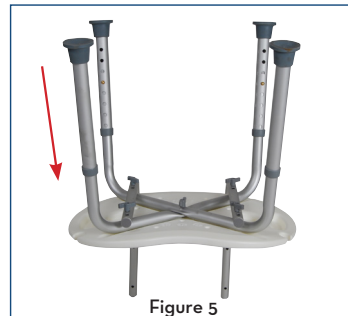
Figure 5

7. Assemble backrest (F) by aligning the backrest hole with the corresponding holes on the backrest frame (E). Secure in place by inserting the short thumb screws (A) and turning clockwise until tight (Figure 6).


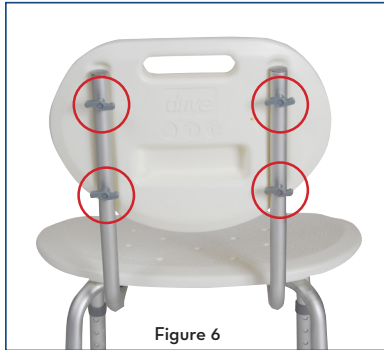
Figure 6

Note: Seat depth can be adjusted by choosing 1 of the 2 holes on the bottom of the frame supports.

## adjusting leg height

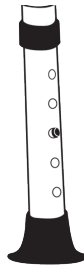1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.
2. To be sure legs are locked into place by pushing down on or by pulling legs.


## additional precautions

1. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Ensure ALL thumb screws are tightened.
4. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
5. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
6. DO NOT stand on chair.

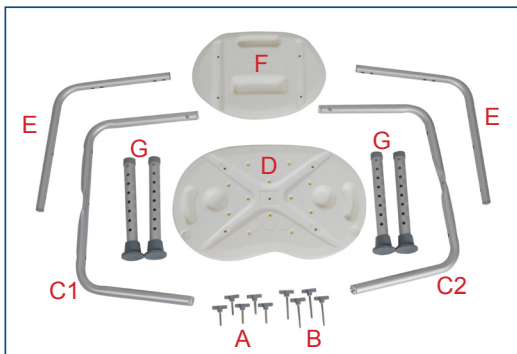NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.