

ASSEMBLY INSTRUCTIONS USE AND MAINTENANCE

FOLDING SEAT

ROME SERIE G25




ANTIMICROBIAL

RECYCLABLE PRODUCT

Thank you for purchasing this Ponte Giulio item. This product has been manufactured in respect of the safety requirements provided by law

> Ponte Giulio USA, Inc W241 S4021 Rockwood Circle Waukesha WI 53189 – USA Phone +1.877.766.8301 * fax +1.877.766.8306 Web Site: www.pontegiuliousa.com Email: pontegiulio@sbcglobal.net

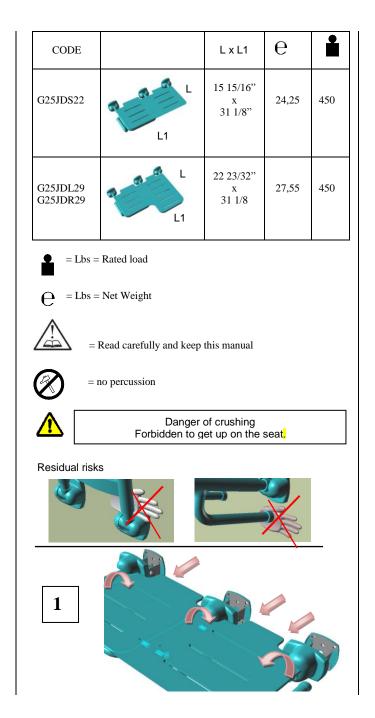


2


3

4


5

6

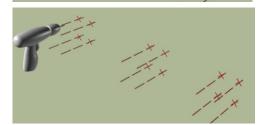

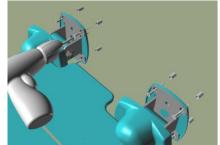



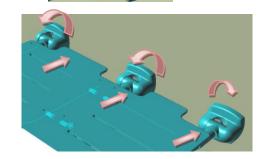

LIMITED WARRANTY

If this product fails due to a defect in material or workmanship during the time of periods listed below for each product, Ponte Giulio USA Corp. will replace or repair it free of charge. Simply contact Ponte Giulio at +1 877 766 8301 for replacement information.

GRAB BAR = 5 YEARS

EXCEPTIONS AND RESTRICTIONS

This warranty does not cover products which have not been installed or used in accordance with the instructions concerning installation and use set forth in the owener's manual or installation instruction, and when the recommendations and warning included are not observed.

This warranty does not cover products that have been modified or dismantled partially or totaly, or have been used in a negligent fashion and as conseguence has suffered damages attributable to the consumer, individual, or hardware not authorized by Ponte Giulio.

This warranty does not cover products which have been abused, damaged, altered, misused, cut or worn, making substitutions or modifying this product in any way will vold your warranty.

This warranty does not cover the damages as a result of disater such as fire or acts of god, including flooding, earthquake, or electric storms, etc.

LIMITATION OF LIABILITY

Other than the limited warranty stated above, to the extent permitted by law, Ponte Giulio disclaims all other implied or express warranties, including all warranties of merchantability and/or fitness for a particular purpose. This warranty is your sole remedy and Ponte Giulio shall not be liable for any damages (whether direct, incidental, special, consequential, exemplary or otherwise, including lost revenues and lost profits) arising out of any theory of recovery, including statutory, contract or tort. Notwithstanding the term of any limited or implied warranty, or in the event that any limited or implied warranty, or in the event that any limited warranty fails of its essential purpose, in no event will Ponte Giulio's entire liability exceed the purchase price of this product, some states and provinces do not allow the exclusion or limitation of incidental or consequential damages so the above limitations or exclusions may not apply to you this limited warranty gives you specific legal rights, and you may also have other rights which vary from state to state and province to province. Ponte Giulio assumes no liability for any personal injury or property damage caused by improper assembly, installation or use of this product.

INSTALLATION

The installation of the safety support must be carried out only by competent and qualified personnel who will have to use the most suitable fixing screws according to the various types of walls. Installers must make sure that the wall by the fixing plate is perfectly flat, without any protruding joints or tiles, that the wall's structure is suitable for fixing the safety support and that there are no weak points. Only use 4 screws for each fixing plate. The omission or non efficiency of one screw will dangerously reduce the safety support's load resistance. The omission or non efficiency of 2 or more screws does not garant the safety support's resistance. The screws in the packaging are suitable exclusively for the following kinds of wall: CONCRETE (>C12/15) FULL BLOCKS IN: GASBETON (>G4) PLASTER. NATURAL LIGHT STONE PERFORATED BRICKS IN: LATERITIOUS (>HLz 12) LIME (>KSL 12) WARNING: excluded from use are perforated blocks in concrete (Hbl) and light concrete (Hbn). SOLID BRICKS IN LATERITIOUS (>Mz 12) LIME (>KSL 12). NATURAL COMPACT STONE, GYPSUM / DRYWALL OVER A WOOD STUD. PLYWOOD BACKER BOARD Before use to verify the correct assembly of the device by simulating the use. The tilting must be only by a slight effort of the user. Periodically check the tightening of the screws as shown in Figure 5

CLEANING AND MAINTENANCE

DO NOT USE abrasive liquid detergents. Chlorine (Bleach) does not improve the aseptic aspect of the equipment and could lead to opacity and deterioration of the material. DO NOT USE hot air with a temperature over $80^{\circ}C$ ($175^{\circ}F$) for drying the device.

DISPOSAL

The user is responsible for disposal, which must be carried out in compliance with the laws of the country of installation. The devices are not manufactured with substances that are harmful or hazardous to health and the environment. All their components are recyclable and/or reusable. In order to protect the environment, the device is manufactured with percentages of recycled materials.

For proper disposal, it is necessary to separate the plastic and Pvc from and place them separately to authorized collection centers.

In case of any doubts, refer to the technical department of Ponte Giulio

The device was designed and constructed to facilitate the use of the bath for the elderly or with disabilities or limited mobility, offering them a solid support

For the correct use the space around the device must be free from any object and easily accessible

For further information, please visit our website or contact the Ponte Giulio technical department

Rev.2 – 16.07.2012